

The Haakaa logo consists of the word "haakaa" in a white, lowercase, sans-serif font, with a registered trademark symbol (®) to its upper right. It is set against a solid black square background.

# SILICONE STORAGE CONTAINER

## PRODUCT MANUAL

English

**Product Name:** Silicone Storage Container  
**Contains:** 1 x 160ml Silicone Storage Container

**Materials:** Container Body - Medical Grade Silicone, PPSU  
Attachment Ring - PP  
Sealing Disk - Medical Grade Silicone

### Features:

- Made with medical grade silicone, with great anti-impact performance.
- Compatible with Haakaa silicone milk bottle and generation 3 breast pump flange for pumping directly into the bottle. No transfer of breast milk is required.
- The twist-tight, leakproof seal makes this container ideal for storing liquids and snacks or taking items on the go.
- Air-tight design to retain breast milk's beneficial properties.
- Large, easy-to-read millilitre and ounce measurements.
- Wide base helps eliminate tip-overs, and the ultra wide-neck design allows for easy cleaning.
- Safe in microwave, boiling water, dishwashers and sterilisers.
- BPA, PVC and phthalate-free.

### Cleaning & Care:

Wash before and after each use. Although dishwasher safe, we recommend hand washing in warm soapy water and rinsing and drying thoroughly. Do not use any bleach-based cleaners or tablets to sterilise or clean your Haakaa products. To sterilise, use a steam steriliser or boil in water for 2-3 minutes.

### Caution:

- Check product condition regularly. Replace if product shows any signs of damage.
- Do not store near any sharp objects.
- Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- This product is not a toy. Do not use for any other purpose.
- Store product in a cool dry place and avoid direct sunlight.

**Please note:** Volume measurements are an approximation and should be used for general reference only.