

MULTIFUNCTIONAL SILICONE BREAST PUMP AND SILICONE PUMP CAP PRODUCT MANUAL

English

Product Name: Multifunctional Silicone Breast Pump And Silicone Pump Cap

Materials: Bottle Body - Medical Grade Silicone, PPSU

Attachment Ring - PP

Breast Pump Flange - Medical Grade Silicone

Cap - Food Grade Silicone

Volume: 5.4 oz. / 160ml, 8 oz. / 250ml

Colours: Pump - Peach / Grey

Cap - Grey

Product Description:

Our famous Haakaa Silicone Breast Pump has been reinvented! We have kept the same simple and effective design that you know and love, except now you can express directly into a detachable silicone bottle! Made with medical grade silicone, our compact pump is perfect for long-distance travel or anywhere you'd like to express discretely, quickly and silently. Simply suction to your breast and let the pump do the work for you! Once you're finished, seal the silicone pump cap over the flange of your pump to prevent accidental spills and keep it sanitary and sterile. The Haakaa Silicone Breast Pump is 100% eco-friendly and is safe for mum, baby and our environment. Our new pump bottle is compatible with all Gen. 3 attachments, and with a quick twist of the top, you can transform your breast pump into a baby bottle, sippy cup, feeding spoon bottle or handy storage container! (Each sold separately.)

Features:

- Pump made with high-quality medical grade silicone.
- Easy and effective milk expression.
- No cords or batteries required.
- Compatible with all Gen. 3 bottle range attachments.
- Pump flange can be replaced with nipple attachment for feeding.
- Easy to use and clean.
- Cap is leakproof to prevent spills and seals tight to keep your pump clean and hygienic.
- Safe in microwaves, sterilisers and boiling water.
- BPA, PVC and phthalate-free.

Cleaning and Care:

Wash before and after each use. We strongly recommend cleaning and sterilising your Haakaa Breast Pump with any steam sterilising system or by boiling in water for 2-3 minutes. Do not use any bleach-based agents or sterilising tablets to clean this product. Do not use UV sterilisers to clean this product as it may impact the lifespan of your pump.

Caution:

- ⊙ Check product condition regularly. Replace if this product shows any signs of damage.
- ⊙ Do not store near any sharp objects.
- ⊙ Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- ⊙ This product is not a toy. Do not use this product for anything other than its intended use.
- ⊙ Store product in a cool dry place and avoid direct sunlight.
- ⊙ Adult supervision is required.

Please note: Volume and ounce measurements are approximations and should be used for general reference only.