



MULTIFUNCTIONAL SILICONE BREAST PUMP

English

Product Name: Multifunctional Silicone Breast Pump

Materials: Bottle Body - Medical Grade Silicone, PPSU
Breast Pump Flange - Medical Grade Silicone
Attachment Ring - PP

Volume: 5.4 oz. / 160ml, 8 oz. / 250ml

Colours: Peach / Grey

Product Description:

Our famous Haakaa Silicone Breast Pump has been reinvented for Generation 3! We have kept the same simple and effective design that you know and love, except now you can express directly into a detachable silicone bottle! Made from medical grade silicone, our compact pump is perfect for long-distance travel or simply anywhere you'd like to express discretely, quickly and silently. Simply suction to your breast and let the pump do the work for you! It's 100% eco-friendly and is safe for mum, baby and our environment. Our new pump bottle is compatible with all Gen. 3 attachments, and with a twist of the top, you can transform your breast pump into a baby bottle, sippy cup, feeding spoon bottle or handy storage container! (Each sold separately.)

Features:

- High quality medical grade silicone.
- Easy and effective milk expression.
- Compatible with all Gen. 3 Silicone Bottle Range attachments.
- Compact, portable and convenient.
- Accommodates different breast sizes.
- Safe in microwaves, sterilisers and boiling water.
- Easy to use and clean.
- Available in 160ml and 250ml capacities.
- BPA, PVC and phthalate-free.

Cleaning and Care:

Wash before and after each use. Although dishwasher safe, we strongly recommend sterilising using a steam steriliser or by boiling in water for 2-3 minutes. Do not use any bleach-based cleaners or tablets to clean or sterilise this product.

Caution:

- ⊙ Check product condition regularly. Replace if this product shows any signs of damage.
- ⊙ Do not store near any sharp objects.
- ⊙ Only soft bristle brushes or soft sponges should be used to clean this product. Hard scourers may scratch the surface.
- ⊙ This product is not a toy. Do not use this product for anything other than its intended use.
- ⊙ Store product in a cool dry place and avoid direct sunlight.
- ⊙ Adult supervision is required.

Please note: Volume and ounce measurements are approximations and should be used for general reference only.